

Roche Resends

Samples are resent if Roche was unable to type the original sample or if we have reason to question the result given. A list of samples to resend is generated each time that new results are imported.

1. Prepare Resend samples before sending out new NEC and NOC samples
2. Label 2mL cryovials with the assigned Roche ID numbers and the initials of the DAISY subject.
3. Locate the cord blood whole blood samples in the lab refrigerator (the list gives the box and row numbers).
4. Transfer 0.5mL of blood from the original sample tube to another tube with the Roche ID.
5. Put the remaining whole blood sample back in its original location.
6. Put the Resend samples with the Roche Resend box located in the lab refrigerator, and record the Roche ID and initials of the subject in the Roche Resend Excel file.
7. The Data Manager will receive the results from Roche and will update them to the database.