

## DAISY Long Distance Protocol

### LAB MOO

On the 2<sup>nd</sup> week of each month, the DAISY Remote Coordinator will add a new LDP track for participants due in the following month.

The coordinator will provide the lab staff with a list of these participants which will include ID, kit type (Regular or LabCorp), and Visit Type (Every year, 6 month, 3 mo, LTF).

Large, pre-addressed manila envelopes with subject IDs will be provided for Regular kits.

Please enter the kit **expiration date** (date on the tube that's expiring first) in the "Notes" section of the "LDPs to send" document.

#### LabCorp Blood Kits

##### **Inside the ThermoSafe/Insulated Box:**

- Red, Purple, and Blue top (pink top for positive kids) – write ID numbers on tubes and place in tube sleeve.
  - Please select tubes with an expiration date at least 3 months out
  - If making kits for siblings, please also write the participant's first name on tubes
- Wrap the tubes in bubble wrap and the biohazard bag and secure with a latex-free rubber band
- Blood draw supplies in a 4x6 bag:
  - 2 Butterfly needles (green)
  - 1 Transfer device
  - 2 Alcohol pads
  - 2 Gauze pads
  - 2 Cotton balls
  - 2 Band-aids

#### Regular Blood Kits

- Red, Purple, and Blue top (gray top for positive kids) – write ID numbers on tubes
  - Please select tubes with an expiration date at least 3 months out
  - If making kits for siblings, please also write the participant's first name on tubes
- Place tubes in styrofoam tube holder w/ absorbent pad, secure with a latex-free rubber band, and place in a plastic biohazard bag
- Label biohazard bag with participant's ID (+ first name for siblings)
- Blood draw supplies in a 5x8 bag:

- 2 Butterfly needles (green)
- 3 Collection syringes
- 1 Transfer device
- 2 Alcohol pads
- 2 Gauze pads
- 2 Cotton balls
- 2 Band-aids

**MAKE SURE ID# MATCHES ON TUBES AND ENVELOPE!!!**