

DAISY LAB

LDP Finger Poke Protocol

1. We will receive 1 or 2 bullets of blood in place of a filter paper for those DAISY subjects that were on the filter paper protocol.
2. Spin the bullets for 10min at 3000 RPM.
3. Aliquot the serum as follows:
 - a. 200ul into the LabID tube
 - b. Then aliquot 200ul into the first Q's tube
 - c. Put the remaining serum up to 1000ul total vol in the LabID tube
 - i. Record the final volume on the clinic visit sheet
 - d. Any remaining serum will be aliquoted into the TG then Q2's tube.