

**DAISY LAB**  
**URINALYSIS-DAISY Positive Keto-Diastix results**

1. Collect urine in sterile specimen cup
2. Dip Keto-Diastix reagent strip into specimen (within 1 hour after collection)
3. Read Strip:
  - a. Ketone-15 seconds
  - b. Glucose-30 seconds
4. If Negative results for both, record results on clinic visit form-test completed
5. If Positive results for either, record results on clinic visit form and in logbook and prepare Positive Control Solution and Negative Control Solution
  - A. 12ml deionized water in tube (x2)
  - B. Place (+) strip in one tube, place (-) strip in another tube
  - C. Cap and invert tubes for 2 minutes
  - D. Allow tubes to stand at room temp for 30 minutes
  - E. Invert tubes once again and remove strips
6. Using Keto-Diastix reagent strips from original bottle, dip one strip into each control solution
7. Read Strips:
  - a. Ketone-15 seconds
  - b. Glucose-30 seconds
8. Record results in logbook in appropriate control fields, initial